




# CLINICAL CONNECTIONS & SETTINGS GUIDE

*JO allows pattern recognition command of Ottobock AxonArm Ergo electric lock with optional AxonRotation and the AxonHook and/or Michelangelo Hand.*

## PART 1: PHYSICAL CONNECTIONS

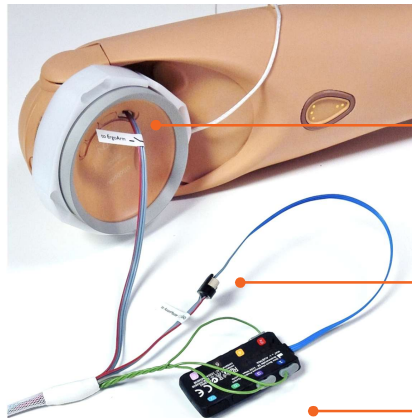

### OTTOBOCK AXONARM ERGO

#### CONNECTION

Coapt cable connection marked '**CONNECT TO Elbow at Input**' must plug into the port on the AxonArm Ergo

#### CONNECTION

Coapt cable connection marked '**CONNECT TO AxonMaster**' must plug into blue/gray cable from AxonMaster

#### CONNECTION

Cables marked '1', '2', and '3' plug into corresponding inputs of the Ottobock AxonMaster

## PART 2: SOFTWARE SETTINGS

### ELBOW

#### OTTOBOCK AXONARM ERGO

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown

### WRIST

#### OTTOBOCK AXONROTATION (OPTIONAL)

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown

### HANDS

#### OTTOBOCK MICHELANGELO HAND, AXONHOOK

Ensure all settings in the Ottobock Data Station 'Customizing' screen are set exactly as shown.

