


K7 allows pattern recognition command of Fillauer Motion E2 Elbow flexion/extension with optional pronation/supination of powered wrists and open/close operation of prosthetic hands. Pattern recognition command of multifunction grips is not included.

PART 1: PHYSICAL CONNECTIONS

FILLAUER MOTION E2 ELBOW

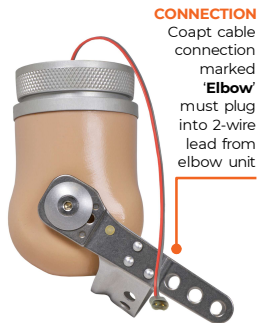

CONNECTION
Coapt cable connection marked 'Elbow' must plug into 2-wire lead from elbow unit


CONNECTION

Coapt cable connection marked 'Close' must plug into this side '2' of wrist

CONNECTION

Coapt cable connection marked 'Rotate' must plug into wrist port marked '3'

OTTOBOCK ELECTRIC WRIST ROTATOR (10S17)

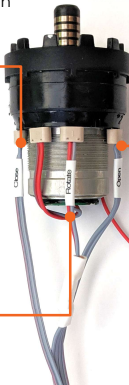


CONNECTION

Coapt cable connection marked 'Open' must plug into this side '2' of wrist

CONNECTION

Power/battery cable (not supplied by Coapt) must plug into '1'

FILLAUER MOTOR DIRECT WRIST ROTATOR (5010045, 5010054, 5010055)

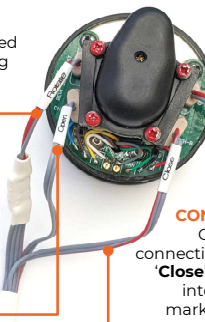

CONNECTION

Coapt cable connection marked 'Rotate' must plug into wrist port marked 'WRIST'

CONNECTION

Coapt cable connection marked 'Open' must plug into wrist port marked 'CH-A'

CONNECTION

Power/battery cable (not supplied by Coapt) must plug into wrist port marked 'HAND'

CONNECTION

Coapt cable connection marked 'Close' must plug into wrist port marked 'CH-A-B'

NOTE:

When using an i-Limb Quantum terminal device, connections for **Open** and **Close** may need to be reversed to ensure proper operation.

PART 2: SOFTWARE SETTINGS

ELBOW

FILLAUER MOTION E2

No settings required

WRISTS

OTTOBOCK ELECTRIC WRIST ROTATOR (NO MYOROTRONIC)

No settings required

FILLAUER MOTOR DIRECT WRIST ROTATOR (NO PROCONTROL)

No settings required

HANDS

OTTOBOCK SENSORHAND, MYOHAND VARIPLUS, SYSTEM ELECTRIC GREIFER, SYSTEM ELECTRIC HAND

Ensure 'control mode'/'program' 1

PSYONIC ABILITY HAND

Ensure the "Control System" setting in the PSYONIC ABILITY HAND app is set to "Direct Control"

Ensure all other hand settings in the PSYONIC ABILITY HAND app are set to the factory defaults

HANDS (CONT.)

OTTOBOCK BEBIONIC

Control Strategy: Dual Site

Open / Close Strategy: Electrodes: A,B

Control Options: Electrodes

Control Response: Proportional In

ELECTRODE SCREEN:

- For both signal channels, set the low thresholds to about 10% of the numbered range and the high thresholds to about 90% of the numbered range

Remember to press "Send to Hand" after making any changes

FILLAUER PROHAND, PROETD/ETD2

HAND SET UP

- Input Type: EMG
- Hand Filtering: Quick
- Channel: Dual
- Control: Differential
- Hand Direction: A

Thresholds/Outputs

- A: 15
- B: 15

Input Gains

- A: 5
- B: 5

ADAMS HAND

Ensure the "Control System" setting in the Adam's Hand is set to "Coapt"

OTHER COMPATIBLE HANDS SHOULD BE SET TO FACTORY DEFAULT FOR OPTIMAL OPERATION